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User Manual:

Manual utilizador:

Handleiding:

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Käyttöohje:

# Total Protection Ag+

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## Visco-elastic foam mattress featuring a welded protection system

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#### TIPS

The first prevention of the pressure ulcers to the bed consists in changing the points of support of the body on the mattress on average every three hours to limit the prolonged compression of the vessels to bony projections and to allow the irrigation of tissues. Do Not overtighten the bed linen thus reducing the effective support of the mattress.

#### INDICATIONS

- For bed patients with reduced mobility (> 15 hours per day) with "Medium to High" risks of pressure sores (On the Braden scale or any other validated scale and on medical advice). Help in pressure sore treatment for stage 1 to 2 pressure sores according to the EPUAP.
- Established pressure ulcers: follow medical recommendations.

#### INSTALLATION

- Position the mattress so that the "arrow + foot" pictograph printed on the upper portion of the mattress is located near the patient's feet. The Viscoelastic layer of the mattress should be on the upper surface.

#### CLEANING / DISINFECTION: • Total Protection Cover Ag+

The delivered removable cover is used to protect the mattress from all contamination by effluents. If the cover is deteriorated it is imperative to change it and not to use the mattress without the cover. The cover can be washed and disinfected in a washing machine at 90°C. The cover can be disinfected using current EC valid disinfectants (do not use corrosive products) The cover must not be ironed or tumble dried. The purpose of the cover is to limit the risk of cross contaminations. Item treated with a biocidal substance that is safe for the user. [www.winnicare.fr](http://www.winnicare.fr)

#### SAFETY

- Limit it to users weighing between: VMA63 : 30 and 150 kg. / VMA64 : 30 and 180 kg / VMA100 : 30 and 150 kg.

#### CARE RELATING TO PROTECTIONS

- Direct contact with injured skin does not pose a risk to the patient: the material has the Class 2 Oeko-Tex® seal of approval.
- Systematically protect the (underlay) mattress by means of a protective covering which is always delivered dry: if the foam has repeated contact with fluid, its properties will be modified regarding supporting the body uniformly and constantly as well as helping to lower pressure values.
- Prohibit scouring, stripping or dissolving products and sharp objects in direct contact with the protective covering.
- Make sure you regularly inspect the normal wear on the protective covering and replace it if it is damaged (holes, cracks, changes in the surface, transparent when looked at in the light).
- The flap over the zipper is designed to reduce to a minimum the risk of liquids and/or micro-particles penetrating inside the mattress; it should not be used as a handle to move the mattress about.

#### CARE RELATING TO MATERIALS

- Avoid prolonged damp contact with the foam.
- Carry out regular visual checks on the condition of the foam: the ALOVA material visibly setting or a non uniform and very slow return are all signs of ageing which compromise the support properties which help to prevent pressure ulcers.

#### STORAGE

- ALOVA supports should preferably be stored flat, out of direct sunlight and excessive humidity.

#### DISPOSAL

- Do not burn or dump the support and its protective covering outside areas intended for this purpose.

#### WARRANTY

The warranty period for ALOVA mattresses (and toppers) is 3 years in normal conditions of use.

The VMA100 mattress has a 2 year warranty for use in normal conditions.

This warranty starts on the date the product is purchased from your reseller. It does not cover normal wear and tear of the product and its protective cover and is not a substitute for legal warranties.

The warranty period for the CIC cover has a separate 4 year warranty in normal conditions of use.

Please contact your reseller and show them the defective product. The reseller will take the necessary steps with our company to either have the repairs carried out or to replace the product.

#### LIFETIME OF PRODUCT

- The estimated lifetime of the Viscoelastic foam is 6 years.

#### COMPOSITION OF MATTRESS

- 43% Viscoelastic Polyurethane foam, 57% High Resilience Polyurethane Foam.

#### SUPPLEMENTARY TECHNICAL INFORMATION

Maximum surface pressures (mm Hg) for indicated patient weight. Measured on a flat (0° tilt) mattress

Reference	Max. Weight	Head	Pelvic	Heel
Mattress VMA64 + C.I.C Cover	180 kg	89	60	75
Mattress VMA63 + C.I.C Cover	150 kg	89	60	75

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